

GTL: The Genetic Testing Laboratory
www.gtldna.com Forensic/Infidelity DNA Test Instructions

Important: Please read entire instruction sheet before beginning. If you need a kit to collect samples from willing individuals, please call 1-866-833-6895. Kits are \$30.00 and are only required if you testing people directly.

TO SEND SAMPLES BY US MAIL

Genetic Testing Laboratory
Genesis Center - A
MSC3ARP, Box 30001
3655 Research Drive
Las Cruces, NM 88003 USA

TO SEND SAMPLES BY FEDEX OR OTHER EXPRESS COURIER
DO NOT USE FOR US MAIL!

Recipient: **John Arroyos**
Company: **GTL Genesis Center A**
Address: **3655 Research Drive**
City: **Las Cruces**
State: **NM** Zip: **88003**
Phone Number: **(505) 646-3465**

Instructions: Print, complete and return this form along with samples and payment to GTL for processing.

step 1: acknowledgement of terms and conditions

I submit these samples willingly and understand that the expert will perform this test in a confidential and professional manner. In the case of a sample submitted for a minor, I acknowledge that the legal guardian or conservator consents to the testing described within this document.

The GTL demands that the information provided to us by you (“the customer”) is correct and accurate. Result of analysis is prepared for the use of the customer to whom it is given or sent. Results can only be released in accordance with the instructions provided by the customer on this document.

The GTL shall not be liable for any loss, or damage suffered by you, or any other person as a result of the test results, or lack thereof.

You warrant that you are legally entitled to possession of the samples you have provided to us. You agree to indemnify us against any loss or damage that we may suffer as result of you providing us with samples which have not been legally obtained.

GTL: The Genetic Testing Laboratory
www.gtldna.com Forensic/Infidelity DNA Test Instructions

I hereby agree to the terms and conditions set out above and confirm the information I have given is true and correct.

Samples Submitted

By:

Printed Name

Signature*

____ / ____ / _____

Signed on (mm/dd/yy)

Address: _____ Home Phone: _____ - _____ - _____

_____ Work Phone: _____ - _____ - _____

_____ Other Phone: _____ - _____ - _____

* Your signature constitutes agreement to General Terms and Conditions that may be viewed at <http://www.gtldna.com/TermsConditions.pdf>

Your Comments:

GTL: The Genetic Testing Laboratory
www.gtldna.com Forensic/Infidelity DNA Test Instructions

step 2: witness identification

Sample collection should be witnessed by a neutral third party with no interest in the test outcome. Failure to do so will mean that the identities of the persons tested cannot be verified, and will not be included on the final report.

Sampling Witnessed

By:

Printed Name

Signature

____ / ____ / ____
Signed on (mm/dd/yy)

Contact Information:

Street Address

City State Zip Code

Phone Number

(____) _____ - _____

Other Contact Info:

Witness Comments:

GTL: The Genetic Testing Laboratory
www.gtldna.com Forensic/Infidelity DNA Test Instructions

**step 3: select a DNA test – First
Sample of Any Type is \$150. Each
additional sample cost is as listed
below.**

Selection	Category	General Comments	Packaging Comments	Charge (Per Sample)
<input type="checkbox"/>	GTL Kit Sample	Buccal or blood samples collected using GTL Collection kit	Ship in packaging provided	\$65.00
<input type="checkbox"/>	Blood (Whole)	Medical blood draw in Vacutainer Tube	Ship in original tube	\$80.00
<input type="checkbox"/>	Blood Stains	Band Aids, tampons, bloody tissues	Ship in paper envelope	\$90.00
<input type="checkbox"/>	Cigarette Butts	2 to 4 butts	Ship in paper envelope	\$90.00
<input type="checkbox"/>	Clothing	Hats, baseball caps, underwear, bandanna	Ship in paper bag	\$90.00
<input type="checkbox"/>	Dental Floss	Do not touch the floss with your fingers.	Ship in paper envelope	\$80.00
<input type="checkbox"/>	Ear Wax	Cotton swab or Q-tip	Ship in paper envelope	\$90.00
<input type="checkbox"/>	Electric Razor Clippings	Place in paper envelope.	Ship in paper envelope	\$110.00
<input type="checkbox"/>	Gum	Do not touch the gum with your fingers. Sugar-free gum is preferred.	Ship in plastic Zip-Lock bag	\$110.00
<input type="checkbox"/>	Hair	Roots or follicles must be attached. Hair from a brush or comb may not work. We need at least 6-10 hairs.	Ship in plastic Zip-Lock bag	\$100.00
<input type="checkbox"/>	Nails (Finger/Toe)	Freshly cut nails work best	Ship in plastic Zip-Lock bag	\$100.00
<input type="checkbox"/>	Razors	Place razor in a paper envelope.	Ship in paper envelope	\$90.00
<input type="checkbox"/>	Sperm/Semen	Condoms, undergarments, clothing, etc... Liquid semen: adsorb suspected semen sample onto a clean cotton swab. Air-dry the swab for 1 hour. Dried semen stains: adsorb semen stain onto a clean cotton swab moistened with distilled water. Air-dry the swab for 1 hour.	Ship in paper envelope	\$100.00
<input type="checkbox"/>	Toothbrush	Air-dry the toothbrush for 30-60 minutes.	Ship in paper envelope	\$110.00
<input type="checkbox"/>	Toothpick	Do not touch the toothpick with your fingers.	Ship in paper envelope	\$75.00
<input type="checkbox"/>	Used Handkerchief	Nasal Mucos. Place in paper envelope.	Ship in paper envelope	\$90.00
<input type="checkbox"/>	Used Kleenex	Nasal Mucos. Place in paper envelope.	Ship in paper envelope	\$90.00

GTL: The Genetic Testing Laboratory
www.gtldna.com Forensic/Infidelity DNA Test Instructions

step 4: notification of results

Please indicate below how you would like to receive your results.

I would like to received my results via:

Telephone – My Password is: _____

Telephone Number: () _____ - _____

Email – Address: _____

Mail – Address:

Attn: _____

Street Address

City,State Zip, Code

If you would like your samples returned to you, please select a shipping option:

FedEx - \$20.00

Priority Mail - \$13.00

step 5: calculate your order total

Please total your amount below.

Example:

1 Article of Clothing: \$150

Compare to Swab Submitted: \$85

Total = \$235

Your Order Total: \$ _____

